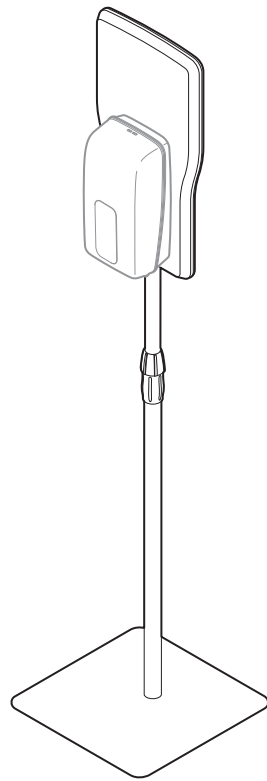
# Adjustable Dispenser Stand IS-9000

## Instruction Manual

Thank you very much for purchasing the Saraya IS-9000 Adjustable Dispenser Stand. You are urged to thoroughly read this instruction manual before operating the IS-9000. Keep this instruction manual handy for future reference.

### parts & Accessories

- Panel ..... 1
- Stand tube ..... 1
- Stand platform ..... 1
- Assembly screws ..... 2
- Dispenser screws ..... 4
- Instruction manual ..... 1

SARAYA

## Safety precautions

Thoroughly read these safety precautions before using the unit. The precautions stated in this manual describe important safety information and you are advised to abide by it. After reading, keep this manual readily available for future reference. For further information, please contact your sales representative.

Definitions	Symbols
<b>Warning</b> Improper handling will result in serious injury (*1) or death.	Never attempt this
<b>Caution</b> Improper handling will result in injury (*2) and/or property damage (*3)	Be sure to follow directions

(\*1) Serious injury herein means vision loss, injury, burning (1st through 3rd degree), electric shock, bone fracture, or poisoning that leaves after effects and requires hospitalization or long term hospital visits for medical treatment.  
(\*2) Conditions herein means injury, burning, electric shock, etc. that does not require hospitalization or long term hospital visits for medical treatment.  
(\*3) property damage herein means extensive damage related to buildings and its contents and domestic animals and pets.

### Installation precautions

**Warning**

Do not use the unit except for its designated purpose. Usage outside the unit's specifications may cause injury or unit malfunction.

Do not install near open flames or near anything that generates sparks. May cause damage or fire. Just in case, keep a fire extinguisher nearby.

Securely position on a strong and level surface. Injuries or damage may result from the unit falling or tipping.

**Caution**

Do not install where directly exposed to sunlight or high temperatures. May cause unit malfunction.

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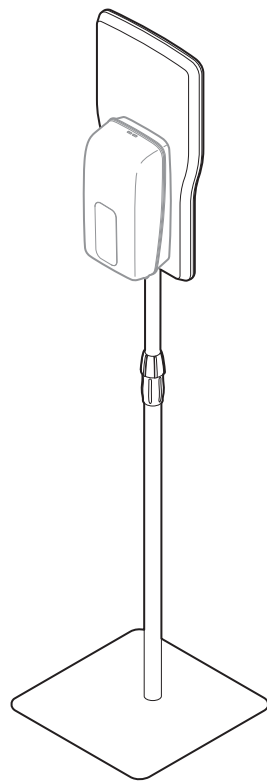
### Usage precautions

**Warning**

Never install this product near sources of open flames such as candles, incense, insect candles, or cigarettes. May cause fire or malfunction.

Use only dispensers approved by Saraya. Other dispensers may cause damage or injury due to falling.

Do not drop or subject to strong impact. Abusive handling may cause malfunction.

Do not pour or splash water on the unit. May cause unit malfunction.

Do not place or hang items on the unit. The unit may cause damage or injury due to falling.

Perform regular maintenance. In order to maintain the unit's safety and quality, perform regular inspections and cleaning.

### Relocation and repair precautions

**Warning**

No one other than an authorized technician should attempt to disassemble, repair or modify the unit. The unit may breakdown. If you have any questions about the unit, please consult your sales representative.

### Disposal precautions

**Caution**

Obey the laws and regulations in your country and area when disposing of the unit.

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## Specifications

Item	Details
Name	Adjustable Dispenser Stand
Model	IS-9000
Dimensions	W350 x D350 x H1310~1700 mm
Weight	Approx. 5.7Kg (excluding packaging)
Materials	Stand platform ..... Painted steel Stand tubes ..... Painted steel Adjustment ring ..... PP Panels ..... ABS
Dispensers	Indicated by Saraya Co., Ltd.

\*Specifications are subject to change without notice or any obligation on the part of the manufacturer.

For questions contact **Saraya Co., Ltd.** 2-2-8 Yuzato, Higashishimiyoshi-ku, Osaka 546-0013 Japan Phone: +81-6-6703-6556

Hours: Weekdays (Except for holidays) 09:00~18:00  
URL: <http://www.saraya.com/>

●Please call for information on using this product and maintenance.

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
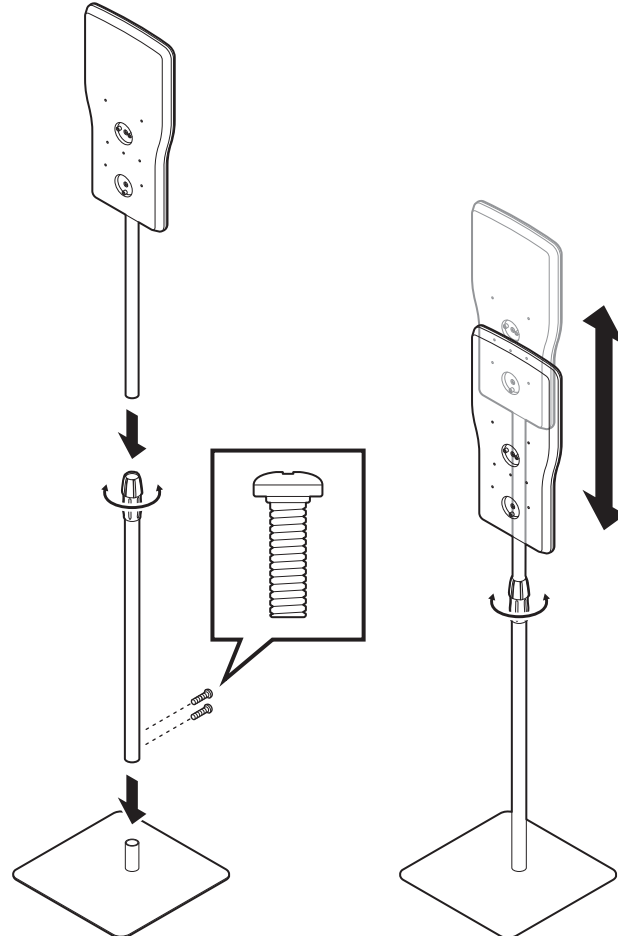
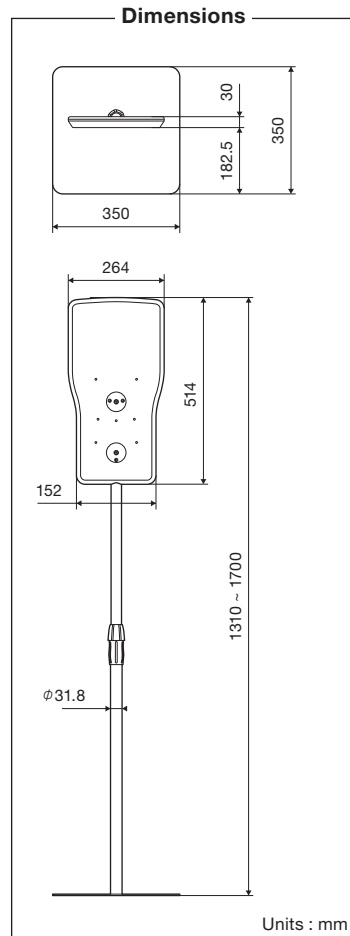
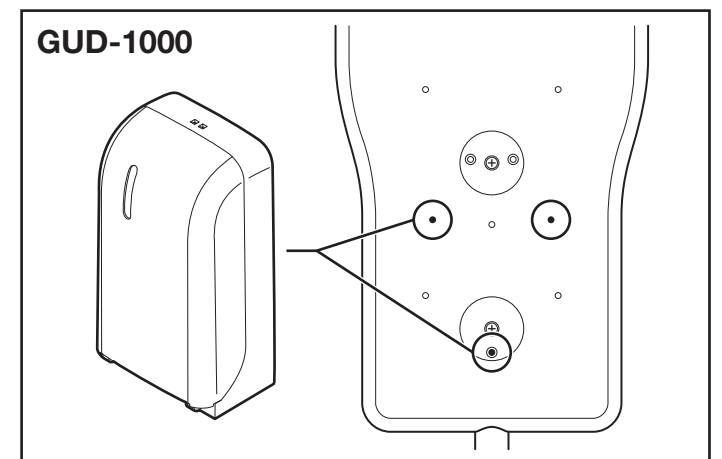
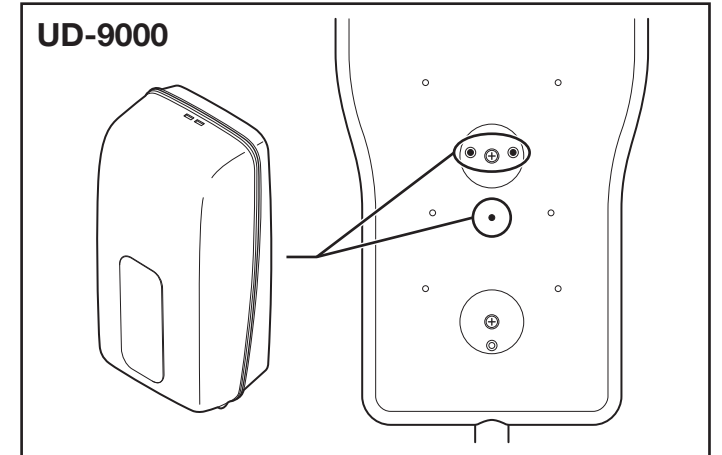
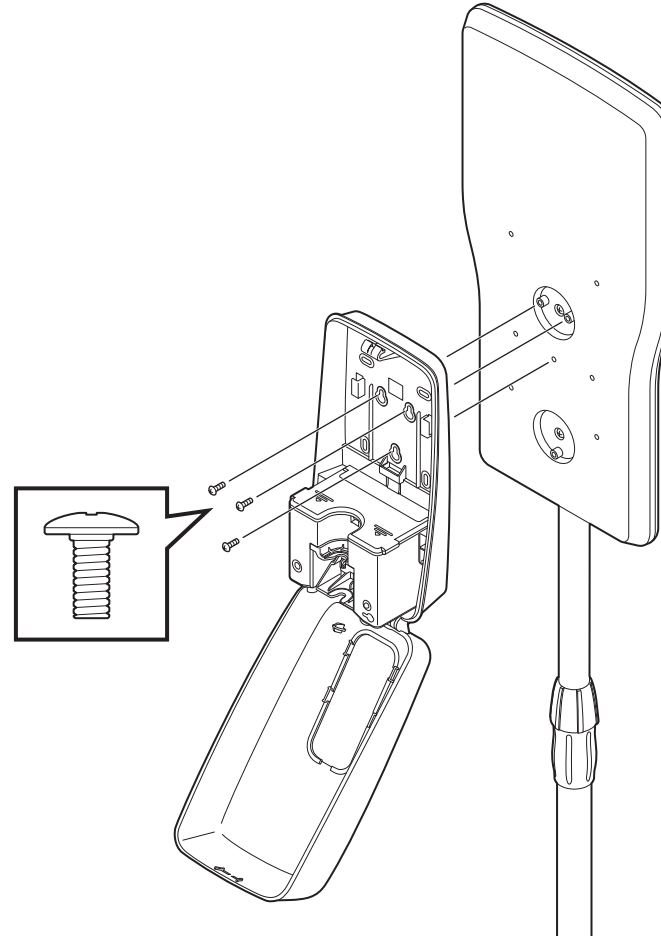
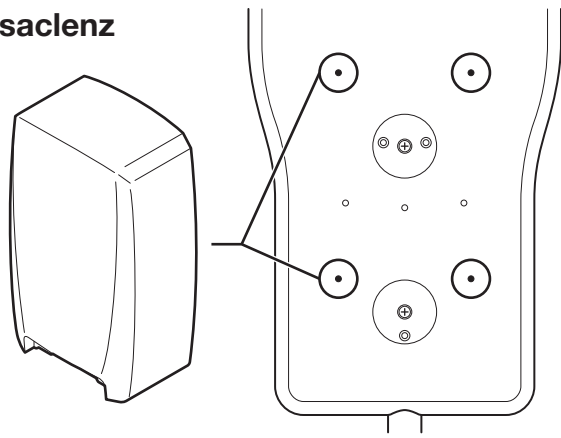
### UD-2200 UD-7500 UD-8600
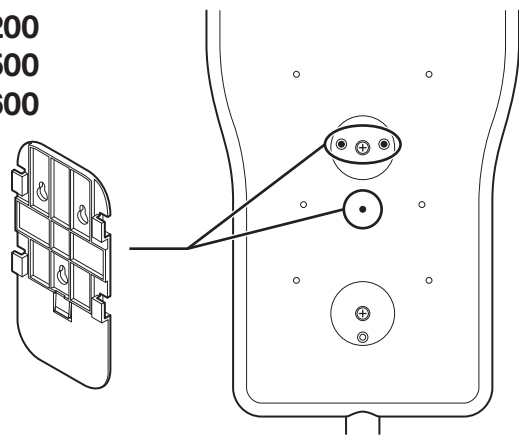
### Message space

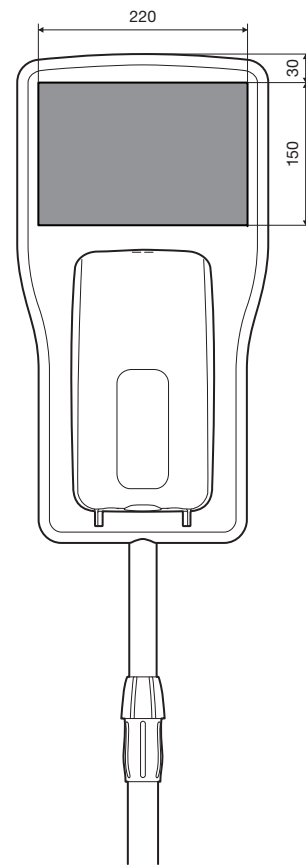
Space on the front panel is provided for attaching messages or posters.

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## 3 Maintenance

Any dirty areas should be cleaned with a dry, soft cloth. Heavy stains should be cleaned with neutral detergent and water, wiped with a moist towel, and dried.

\*Do not use thinner, benzene, petroleum, powder soap, polishing powder, non-neutral detergent, boiling water, acids, alkaline materials, brushes, etc. to clean the surface.

Follow the instructions when using a chemically pre-treated cloth.

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